

InterDry® with FourFold Technology™ simultaneously
targets all four causes of intertrigo¹



Moisture
Wicks and translocates
moisture away from
the skin with 100%
polyester fabric



Friction
Reduces friction with
polyurethane coating



Bacteria
Fights bacteria with
antimicrobial silver in
the fabric




Fungus
Fights fungus with
antimicrobial silver in
the fabric

How to use InterDry:


Scan or text to view “how to” video:
Scan QR code below using your smart phone camera. Launch video by tapping
the link that appears on your screen (not compatible with all smart phones).




Or, text the word **interdry**
to **94253** to view video.

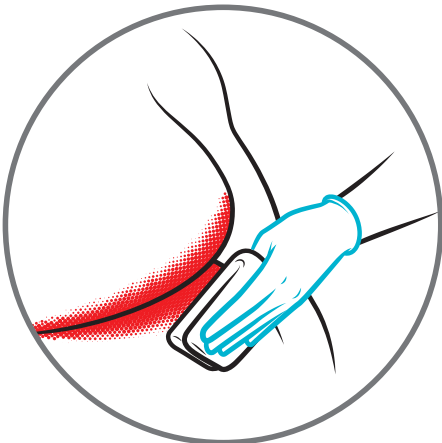


1. Wash skin gently. Pat dry, do not rub.

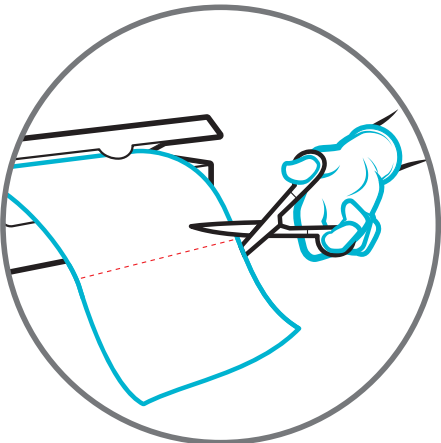


2. Cut the appropriate size of fabric with
clean scissors. Allow a minimum of
2 inches of the fabric exposed outside
the skin fold for moisture evaporation.

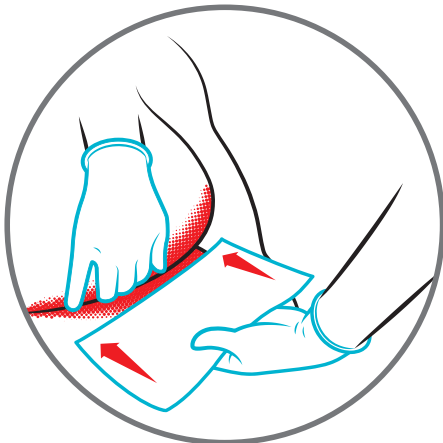


3. Lay a single layer of fabric in the skin
fold, placing one edge into the base of
the fold. Gently smooth the rest of the
fabric over the skin, keeping it flat.

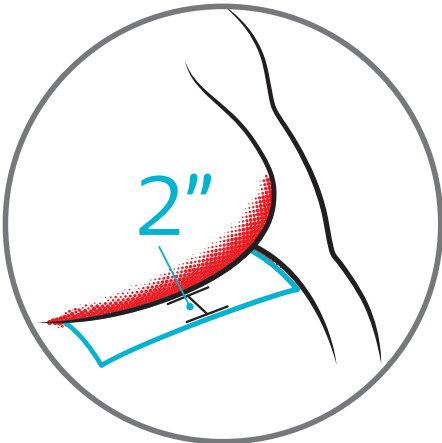

4. Leave at least 2 inches of fabric
exposed outside the skin fold.

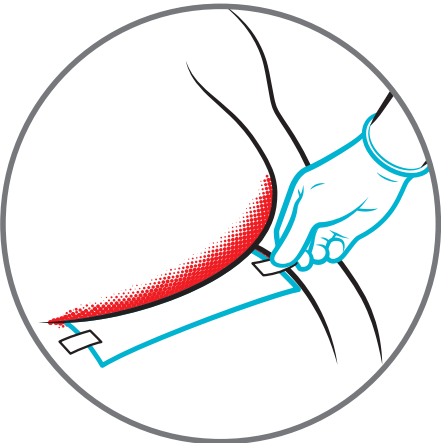

5. Secure fabric with one of several ways:
natural pressure between the skin fold
or device, with a small amount of skin-
friendly tape, or tucked under clothing.

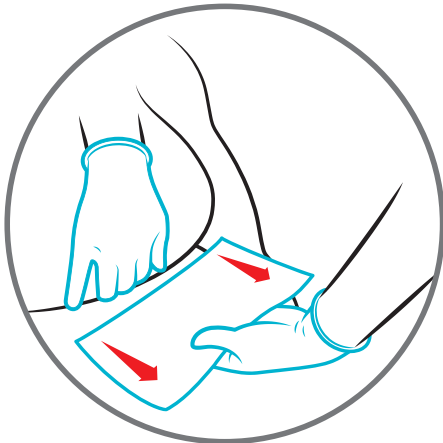

6. Remove the fabric before bathing and
reuse as long as the fabric is not soiled.
(e.g. creams, urine, and feces). When
removing the fabric from the skin folds,
gently separate the skin fold and lift
away the fabric.

Refer to product package for complete instructions including indications, precautions, warnings, storage, and other information. If skin irritation does not improve or worsens over time, talk to a physician or other health care professional. For further questions, call Coloplast Corp. at 1-800-533-0464 and/or consult the company website at www.coloplast.us.

Application sites:

Skin-to-skin

neck roll

axilla

breast

back

abdomen

between fingers

groin

back of knee

between toes

Skin-to-device

trach ties and
cervical collars

blood pressure cuffs

central lines

slings

arm casts

heel suspension devices,
compression garments,
and removable air casts

Helpful hints:



5 days

Wear time depending on fabric
soiling, odor, amount of moisture,
and general skin condition



2 inches

of InterDry® fabric left exposed
outside skin fold or medical device



No creams

ointments or powders as it may
interfere with products efficacy

Ordering Information

Coloplast Customer Service **1-800-533-0464**

Code	Size	Units
7910	10 X 144 in (25 x 366 cm)	10 rolls/cs
7912	10 X 36 in (25 x 92 cm)	10 pks/box
7915	10 X 18 in (25 x 46 cm)	40 pks/box

1. Kennedy-Evans KL, Viggiano B, Henn T, Smith D. Multi-site feasibility study using a new textile with silver for management of skin conditions located in skin folds. Poster presented at: 20th Annual Symposium on Advanced Wound Care; April 28 - May 1, 2007; Tampa, FL and 39th WOCN® Society Annual Conference; June 9-13, 2007; Salt Lake City, UT.